

Maymom IBCLC Starter Kit

Thank you for choosing Maymom IBCLC Starter Kit (ISK). We are willing to help anyone who intends to become a lactation consultant. This kit contains connectors and shields of many sizes for finding the best fit for the mothers. Also, there are various adapters which can combine breast pump parts together even from different brand makers. Following is detailed information:

Kit Contents

Model No. IBCLC-StarterKit_B

PART 1. Base Connectors and Valves & Membranes

PART 2. Maymom MyFit Shields and Flange Inserts

PART 3. Adapters

PART 4. Maymom Pump Kit Set for Spectra Pump

PART 5. Other tools

- Nipple Ruler Card – for measuring moms' nipple diameter
- Colostrum Collection & Storage Bottle
- Feeding Cup – for baby needing short-term feeding

Cleaning Guide

It is important to ensure the parts are clean and disinfected, especially when different users and human milk contact the parts. Made of various materials, each accessory has its own cleaning method. Please refer to the respective instructions and follow them accordingly.

Before use and once daily

- **DO NOT wash the Tubing**
- Disassemble all other parts, clean with soapy water and rinse in warm water.
- Boil the parts in boiling water for 3-4 minutes.

After every use

- **DO NOT wash the Tubing.** Air dry. Replace the Tubing if it gets dirty.
- Disassemble, clean and rinse the parts in warm water.
- Replace the duckbill if it has a gap more than 0.5 mm (thickness of 5 pieces of copy paper) at the lip area.



Make sure you have plenty of water in the pot.



The rim of the pot without water will become over heated and it will melt the plastic.

Base Connectors

PART 1

- Separable connector for breastshield.
- Maymom MyFit Shields from PART 2 are needed to make the connection.



Model No. M012-tBx2

- **Narrow-neck** base connector.
- Top air path type.
- Compatible with Medela Shields, Maymom shields.
- Suitable for Medela pumps made prior Jun 2020 - Pump in Style, Swing, Lactina, and classic Symphony.



Model No. A012-Bx2

- **Narrow-neck** base connector.
- Interchangeable and compatible with Ameda breast flanges.
- Suitable for Ameda pumps: Purely Yours, Purely Yours Ultra, Elite, Mya Joy, Finesse, and Platinum.
- Compatible Pumping Bra: Simple Wishes, Medela, BRAVADO, Pump & Nurse, Pacasso, and Pump Strap.



Model No. S012W-Bx2

- **Widemouth** base connector.
- Compatible with Medela PersonalFit Shields, Pumpin' Pal Breast Pump Flange, Lansinoh ComfortFit Breast Pump Flanges, and Maymom MyFit shields 10-36 mm.
- Suitable for Spectra pump S1, S2, 9 plus.
- Handsfree pump bra friendly.
- Works with Spectra wide mouth bottles, Avent bottles, and Maymom wide mouth bottles.



Model No. M112-BDMBfpSc.x2

- **Narrow-neck** base connector with duckbill valve.
- Built-in backflow protector.
- Compatible with Medela PersonalFit Shields, Pumpin' Pal Breast Pump Flange, Lansinoh ComfortFit Breast Pump Flanges, and Maymom MyFit shields 10-36 mm.
- Suitable for Medela Pump in Style with MaxFlow, classic Freestyle, Freestyle Flex, Swing Maxi, Sonata, Spectra S1/S2/9 plus.

Valves & Membranes



Model No. M005-4V6M

- Compatible with Medela Breast pumps: Swing, Lactina, Pump in Style.

Notice - DO NOT let the breastmilk dry out between the valve and breastshield. Dried milk can act like glue!

Maymom MyFit Shields

PART 2

- Use with Base Connectors from PART 1.
- Compatible with Medela breast pumps: Pump in Style with MaxFlow, classic Pump in Style, Lactina, Symphony, Swing, Mini Electric, Freestyle, Swing Maxi, Sonata, Freestyle Flex and Harmony.
- Easy to detach and attach for thoroughly cleaning.
- Work with all hands-free pumping bras.

Standard type
(13 sizes in total)



Model No. MP012-F13x2
MP012-F15x2
MP012-F17x2
MP012-F19x2



MP012-F21x2
MP012-F23x2
MP012-F24x2



MP012-F27x2
MP012-F28x2
MP012-F30x2
MP012-F32x2
MP012-F34x2
MP012-F36x2

Crater Series
(8 sizes in total)



Model No. MP012C-F10x2
MP012C-F12x2
MP012C-F14x2
MP012C-F16x2




MP012C-F18x2
MP012C-F20x2
MP012C-F22x2
MP012C-F24x2

- Created for a better fit with **bowl-shaped** breasts.
- Less void space between the areola and the shield.
- Offering more comfortable and efficient pumping.
- Long nipple tunnel for elastic nipples.

Flange Inserts

PART 2

- Designed to reduce the internal diameter of nipple tunnel from regular size down to smaller ones.
- Two series of flange insert, classic and shorty, have been developed to meet different users' needs.

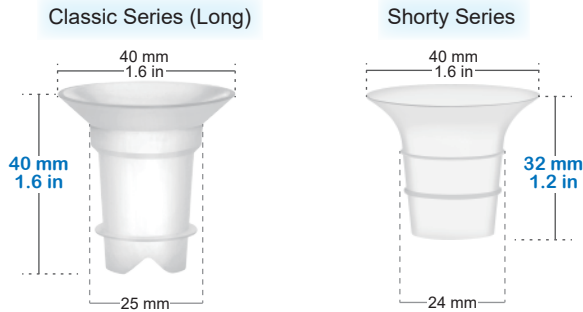

Model No. X012-i10x2
 X012-i13x2
 X012-i15x2
 X012-i17x2
 X012-i19x2
 X012-i21x2

- **Classic Series (Long)** - 6 sizes in total
- Designed to reduce flange sized 24/25 mm down to 10/13/15/17/19/21 mm.
- Compatible with Maymom, Medela and Spectra 24 mm shields/flanges, Willow 24mm flanges, and Momcozy S9/S10/S12 Wearable Breastpump 24 mm cups.
- Fits Freemie 25 mm cups and Lansinoh 25 mm flanges.
- Compatible with Ardo 26 mm flange.



Model No. E012-i13x2
 E012-i15x2
 E012-i17x2
 E012-i19x2
 E012-i21x2

- **Shorty Series** - 5 sizes in total, 4 colors available
- Designed to reduce breast shield sized 24 mm down to 13/15/17/19/21 mm.
- Compatible with Elvie Single/Double Electric and Elvie Stride 24 mm cups, Momcozy S9/S10/S12 Wearable Breastpump 24 mm cups, Legendairy Milk 24 mm Collection Cup, and Tsrete 24 mm Wearable Silicone Cup.
- Fit Medela PersonalFit Flex Breast Shield, Motif Luna flange, Ameda Mya flange, Ameda Mya Pro 24mm flange.

Adapters


PART 3




Model No. M001p-2

- Connects between a Medela PIS MaxFlow/Freestyle Flex connector and Spectra pump S1/S2. (See graph 1-1)
- Connects between a Maymom narrow neck flange/connector and a bare end tubing. (See graph 1-2)

1-1

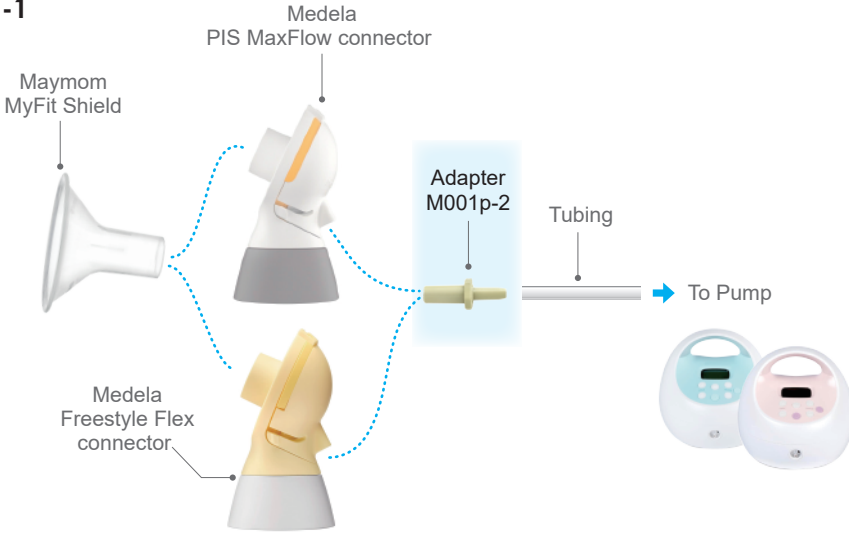

1-2

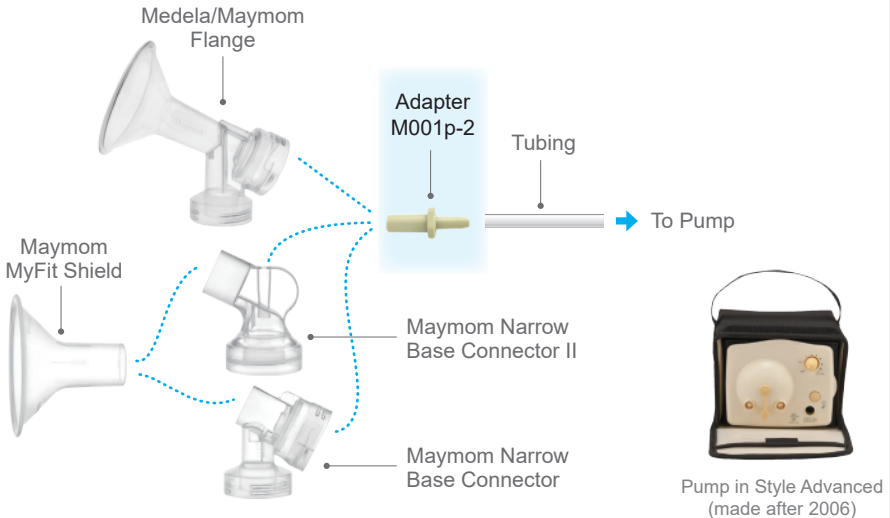

Adapters


PART 3




Model No. M030p-2

- Connects between a Medela Sonata connector and Spectra pump S1/S2. (See graph 2-1)
- Connects between a Maymom widemouth flange/connector and a bare end tubing. (See graph 2-2)

2-1

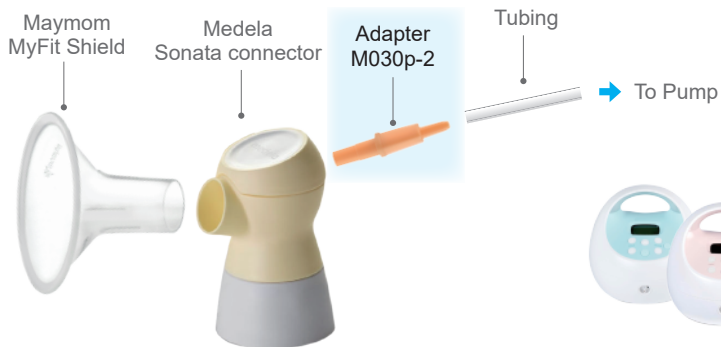

2-2

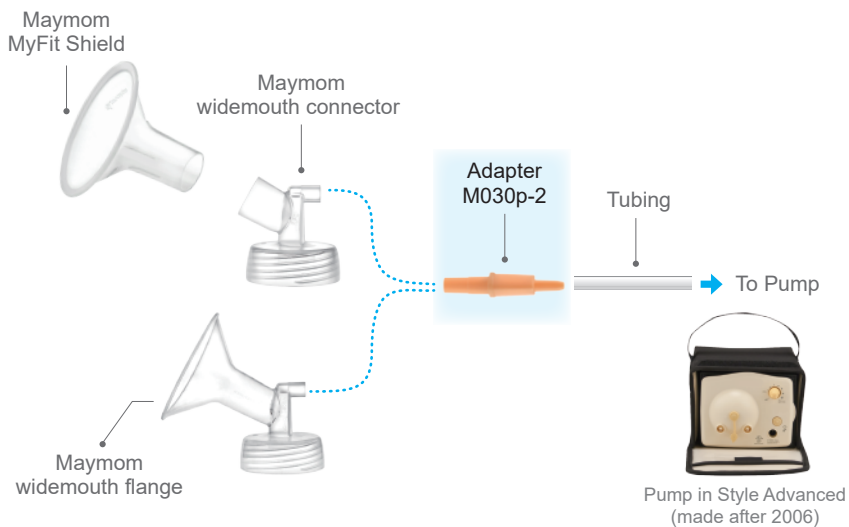





- Connects between a Medela Lactina, classic Symphony, Swing, as well as Pump in Style Advanced (made between 2004 ~ July 2006) pump and a bare end tubing. (See graph 3-1)
- Connects between a Medela Swing Maxi pump and a bare end tubing. (See graph 3-2)

Model No. M002p-2

3-1

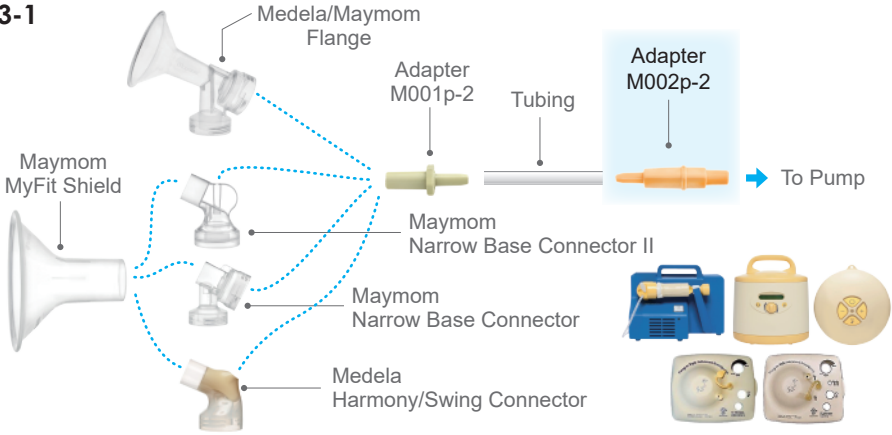

3-2

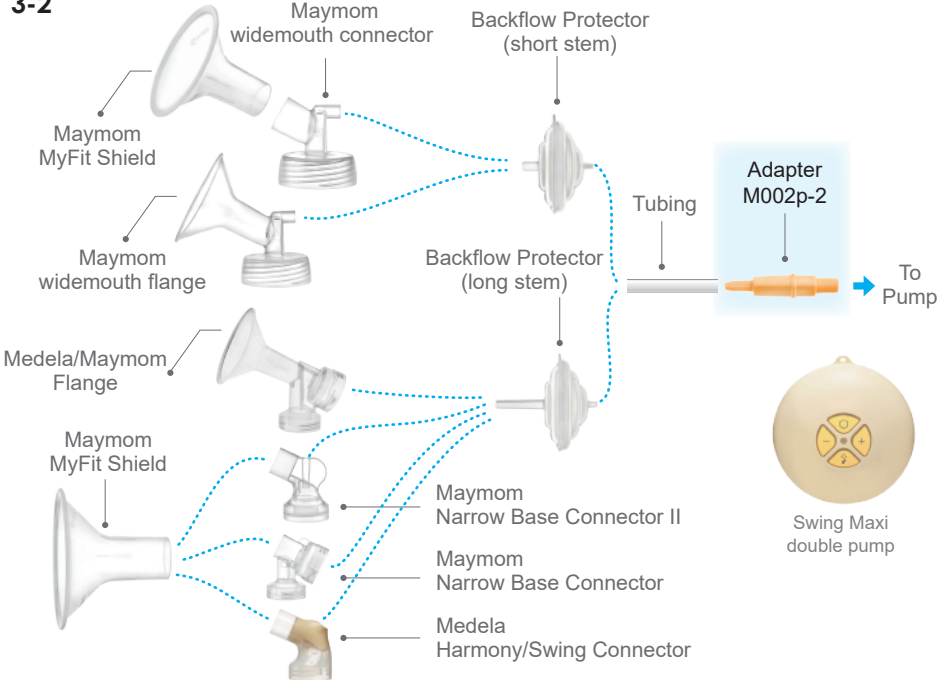

Adapters


PART 3




Model No. M011p-2

- Connects between Medela Swing Maxi connector and Spectra pump S1/S2. (See graph 4-1)
- Connects between a Medela classic Freestyle (old version) connector/pump and a bare end tubing. (See graph 4-2)

4-1

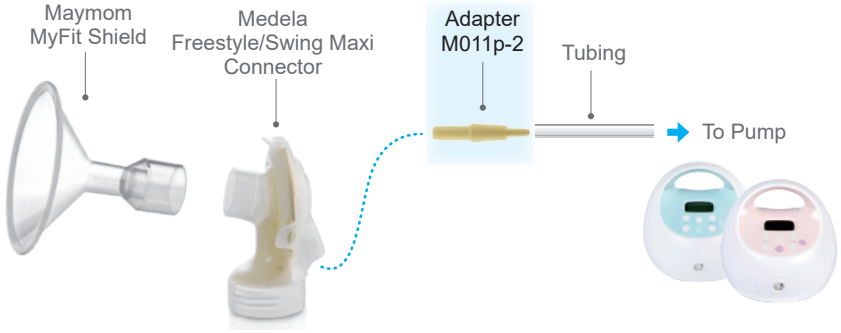


4-2

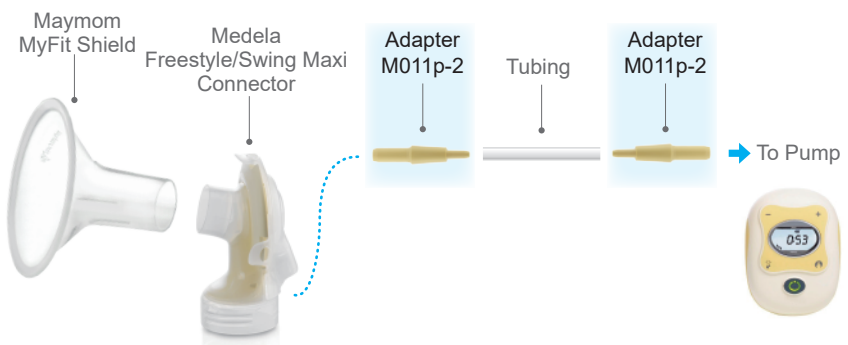

Model No. S012p-2

- For using Medela flanges with Spectra pump S1, S2, M1, 9 Plus. (See graph 5-1)

5-1

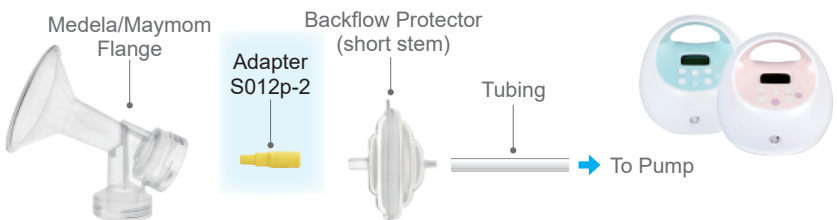





- Can be used in many applications where 3.2 - 4.0 mm size change to 6.2 - 8.5 mm is needed. (See graph 6-1 & 6-2)
- Size changers for Freemie cups to be used with other pumps. (See graph 6-2)

Model No. FP002-4Sc

6-1

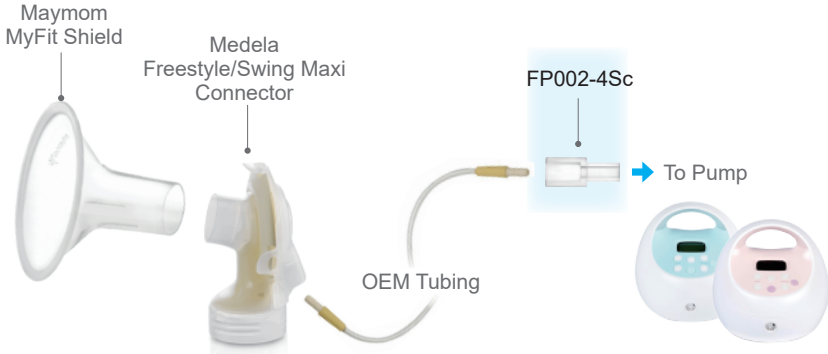

6-2

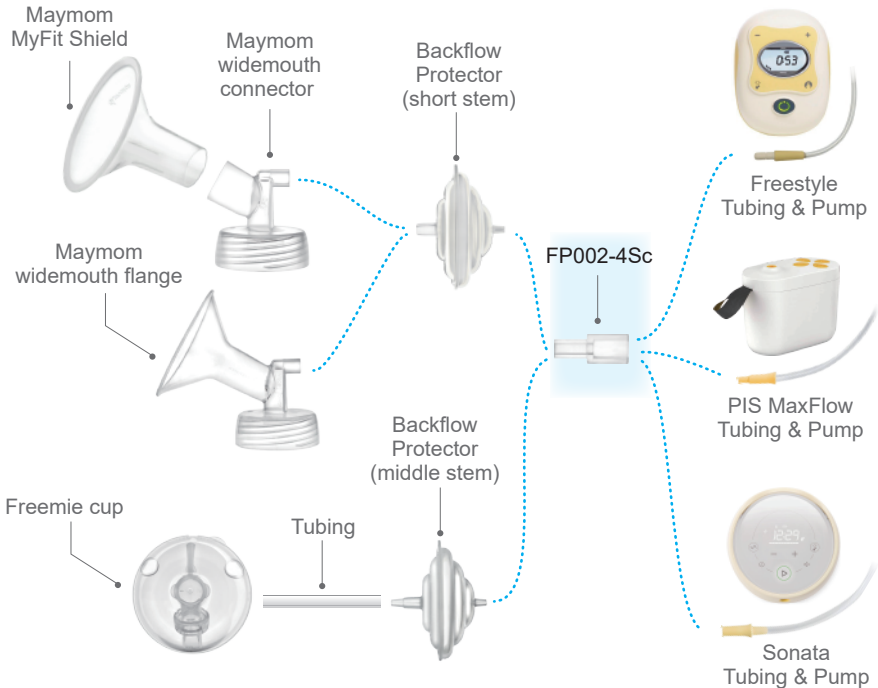


Maymom Pump Kit Set for Spectra Pump



PART 4



Model No. S001W-F24.5in1B

- Work with Spectra pump S1, S2, 9 Plus
- This kit includes:
 - 2 One-piece Flanges (24 mm)
 - 2 Duckbill Valves
 - 2 Backflow Protectors (short stem)
 - 2 Tubes
 - 2 Bottles with One-piece Travel Caps
- Maymom Widemouth One-piece Flange are available in those sizes: 13, 14, 15, 16, 17, 18, 19, 20, 21, 23, 24, 25, 27, 28, 30, 32 mm
- Maymom baby bottle fits Avent Pump ISIS, Avent Comfort Single/Double Electric Breast Pump, Avent Comfort Manual Breast Pump, Spectra widemouth flange, Spectra collar and nipple, Avent Nature collar and nipple, Avent Anti-colic collar and nipple.

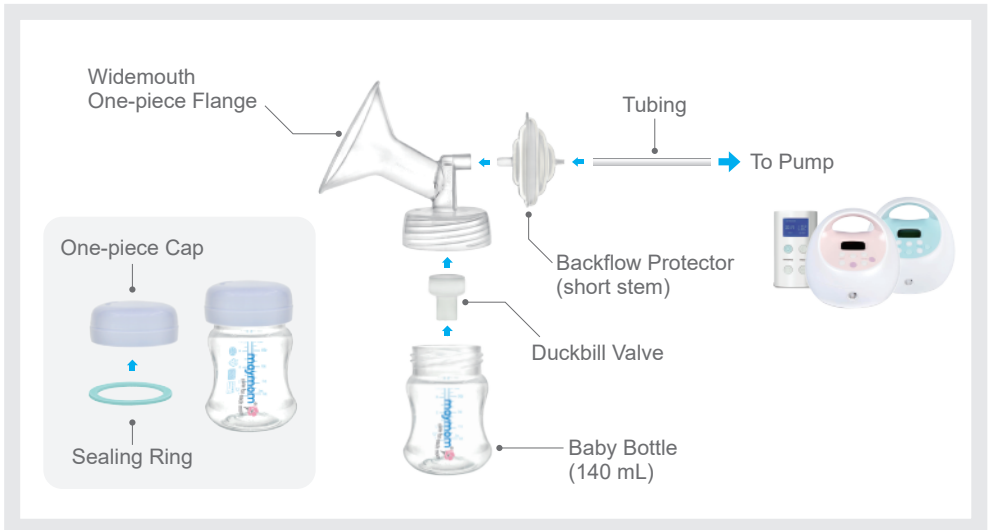

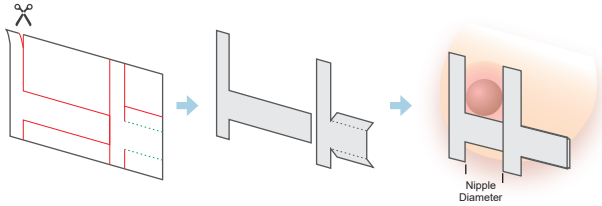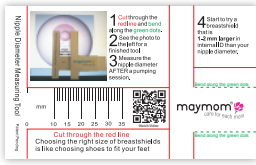

- A tool for moms to measure their nipple diameter.
- Remember to measure the nipple diameter **AFTER** a breast pumping session. Choose a breastshield with an internal diameter that is 1-2 mm larger than your nipple diameter.
- If you measure **BEFORE** a pumping session, you can only estimate to get one that is close. Try one that is 3-5 mm larger than your nipple diameter.

Colostrum Collection & Storage Bottles



Model No. M016-Bt-45x2

- Capacity of 45 mL (1.5 Oz).
- Sloped bottom designed to allow for accurate and easy to read 0 - 5 mL volume.
- For measuring small amount gold liquid like colostrums.
- Screw-on lid provides leak-proof seal and is also compatible with Medela containers.
- Compatible with Maymom narrow-necked breastshields (all models), and Ameda pumps (selected models like Mya Joy, Purely Yours).

Feeding Cup



Model No. X225-4Cup





- Easy to track the small amount of feeding.
- For babies requiring medication or supplement.
- For premature babies.
- Babies with sucking disorder such as cleft lip/palate children.
- Alternative feeding method to bottle or spoon feeding.

Contents List

Model No. IBCLC-StarterKit_B

Model No.	Quantity	Model No.	Quantity	Model No.	Quantity
A012-Bx2	1 pack	MP012C-F14x2	1 pack	MP012-F32x2	1 pack
E012-i13x2	1 pack	MP012C-F16x2	1 pack	MP012-F34x2	1 pack
E012-i15x2	1 pack	MP012C-F18x2	1 pack	MP012-F36x2	1 pack
E012-i17x2	1 pack	MP012C-F20x2	1 pack	M112-BDMBfpSc.x2	1 pack
E012-i19x2	1 pack	MP012C-F22x2	1 pack	S001W-F24.5in1B	1 pack
E012-i21x2	1 pack	MP012C-F24x2	1 pack	S012p-2	1 pack
FP002-4Sc	1 pack	MP012-F13x2	1 pack	S012W-Bx2	1 pack
M001p-2	1 pack	MP012-F15x2	1 pack	X012-i10x2	1 pack
M002p-2	1 pack	MP012-F17x2	1 pack	X012-i13x2	1 pack
M005-4V6M	1 pack	MP012-F19x2	1 pack	X012-i15x2	1 pack
M011p-2	1 pack	MP012-F21x2	1 pack	X012-i17x2	1 pack
M012-tBx2	1 pack	MP012-F23x2	1 pack	X012-i19x2	1 pack
M016-Bt-45x2	1 pack	MP012-F24x2	1 pack	X012-i21x2	1 pack
M030p-2	1 pack	MP012-F27x2	1 pack	X225-4Cup	1 pack
MP012C-F10x2	1 pack	MP012-F28x2	1 pack	Nipple Ruler	20 pc
MP012C-F12x2	1 pack	MP012-F30x2	1 pack		

Maymom Customer Service

 CS@maymom.com
 <http://maymom.com/>
 +1 919 4535168
 Maymom, LLC
 3 Laurel Leaf CT Durham, NC 27703




 care for each mom™

Maymom always cares about mothers' needs on the path of breastfeeding. For years, we hear mothers' voice for customized, comfortable and easy-to-use tools. Maymom products feature safety, efficiency and credibility that you can count on.



Find more!

*Ameda and its pump models are registered trademarks of Ameda INC. Corp.

*ARDO is a registered trademark of Ardo Medical AG.

*Avent is a registered trademark of Koninklijke Philips N.V..

*Elvie is a registered trademark of Chiaro Technology Ltd.

*Freemie is a registered trademark of Dao Health.

*Legendary Milk is a registered trademark of Legendary Milk, LLC.

*Lansinoh is a registered trademark of Lansinoh Laboratories, Inc.

*Medela and its pump models are registered trademarks of Medela Holding AG, Switzerland.

*Momcozy is a registered trademark of Shenzhen Lutejiacheng Technology Co., Ltd.

*Motif and its pump models are registered trademarks of Motif Medical, LLC.

*Pumpin' Pal is a registered trademark of Indeco LLC

*SPECTRA and the SPECTRA Logo are registered trademarks of Mother's Milk, Inc. DBA Spectra Baby USA and Min Byung Wook DBA Uzinmedicare.

*TSRETE is a registered trademark of Minquan Zhiyue Trading Co., Ltd.

*Willow is a registered trademark of Willow Innovations, Inc.

Maymom is not affiliated with them nor do they endorse Maymom's products and compatibility claims.